ENA Soft

(EN) ENGLISH ENA Soft instructions	Page 2
(IT) ITALIANO ENA Soft istruzioni	Pag. 3
(DE) DEUTSCH ENA Soft Gebrauchsanweisung	Seite 5
(FR) FRANÇAIS ENA Soft instructions	Page 7
(ES) ESPAÑOL ENA Soft instrucciones	Pág. 9
(PT) PORTUGUÊS ENA Soft instruções	Pág. 11
(EL) Ελληνική ENA Soft οδηγίες	σελίδα 12
(CS) Český ENA Soft Návod na použití	Str. 14
(PL) Polski ENA Soft Instrukcje	Str. 15
(RU) РУССКИЙ ENA Soft инструкция по применению	Стр 17
(HU) Magyar ENA Soft Használati utasítás	Oldal 19

ENA Soft

(EN) **ENA Soft**

ENA Soft is a light-curing temporary filling material. ENA Soft is an elastic material for temporary restorations. It has advantages of easy removal in one piece without damage to preparation. ENA Soft is suitable also for long term temporaries because it perfectly maintains the shape of tooth.

ADVANTAGES

- simple to apply: not sticky;
- easy to remove in one piece;
- suitable for inlay & onlay and filling on implants screws;
- strong and long lasting fillings.

Composition

- monomer matrix: 30%-70% UDMA, TEDGMA
- filler: 10% - 50% silicone dioxide
- pigments: 1% - 10%

Indications ENA Soft is used for:

- temporary fillings on inlay & onlay
- fillings on implant screws

Contra-indications

If a dry working field cannot be established or the stipulated application technique cannot be applied. Uncured resin could cause skin allergy: user should use gloves. In case of known allergy to some of the components do not use it.

Materials to be avoided

Materials containing phenolics (like eugenol) could inhibit composite curing. Avoid the use of these materials as liners.

Precautionary Information

- The product contains substances that may cause an allergic reaction by skin contact in certain individuals.
- To reduce the risk of all allergic response, minimize exposure to uncured materials. If prolonged contact occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed.

Instructions for use

- 1) Prepare the cavity;
- 2) Application of ENA Soft;
 - a. Carefully clean and dry the cavity to be restored before applying Ena Soft. In case a cavity is without external support, we suggest to use a matrix.
 - b. Remove the cap from the syringe. Turn the extruder clockwise to

dispense a sufficient quantity of material.

c. Apply Ena Soft in the cavity using a composite instrument.

d. Check occlusion and then remove excess material.

3) Light-cure for 10-20 seconds, increments of 4 mm thickness, using a led or halogen light-curing unit (minimum 650 mW/cm²);

4) Remove the matrix if used.

The restoration can be removed with an instrument: it comes out in one piece.

Use and storage

Do not store below 3°C/38°F and above 25°C/77°F. Shelf life at room temperature is 36 months. Ambient temperatures routinely higher than 25°C/77°F may reduce shelf life. Do not use the product after the expiration date (see label on syringe). Use the material at room temperature. Medical device, for dental use only: keep away from children. Turn back the spindle after taking out the material, to avoid sticking of the material, and close the container. After use, close container with cap and keep it closed. Avoid direct exposure to sunlight.

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