

ENAMAL ORAL CARE



MICERIUM S.p.A.
Via Marconi, 83
16036 Avegno (GE) Italy
Tel. (+39) 0185 7887 880
Fax: (+39) 0185 7887 970
www.enacare.it
sales@enacare.it

ENAMAL ORAL CARE




DB digital brush
chlorhexidine

Disposable towel. Sanitizing & antiseptic with Chlorhexidine

Simply wrap the towel around your finger, run it on the tooth surface and on the gums, after every meal, when you are outdoors (as an alternative to the toothbrush), to remove food debris and bacteria, helping to prevent dental plaque.

- Antiseptic properties
- Essential when you can't brush your teeth:
- -oral hygiene outdoors
- -post-operative oral hygiene and in the presence of aphthae
- It removes pigmentations caused by food,
- preserving the white of your teeth.

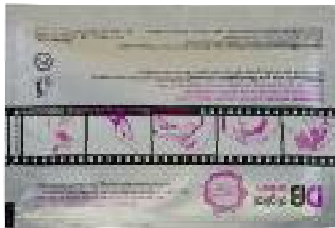

How to use the towel!
Step by step: make a vertical movement, from gum to tooth, both on the upper and lower teeth. Repeat this movement also in the internal side of the mouth.

ECDB1 10 pcs

ECDB18 50 pcs

Included into the MISS ME KIT

Life begins without bacteria in your mouth...


Learning to clean your teeth as a child helps to prevent caries, gingival infections and also to maintain good oral health for all your life



Patented system. Sterilized with β-rays to avoid the use of preservatives.

ECDB2 10 pcs

ECDB28 50 pcs

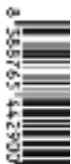

- No contraindications
- Ideal for frequent use
- Recommended for babies
- Ideal for baby-teeth and gums
- Ideal for people with limited manual ability
- Ideal for caregivers

Disposable towel. Health and safe hygiene from birth

Some characteristics of Digital Brush with Chlorhexidine but soaked in Saline.

DB digital brush
saline

Kiss Me Kit

The kit that enhances your smile... wherever you are!

At work, during your free time, to feel always self-confident, keep the Kiss Me Kit in your pocket!



ECKMK 5 Digital Brush towels – 1 Interdenal Brush
1 Interdenal Floss with mirror – 1 Oral Spray

Kiss Me oral spray

Pocket mouthwash

Against bacteria and bad breath

- Ideal formulation against bacteria
- Effective against halitosis
- Cool mint flavour
- Gas and alcohol free
- Spray it directly into the oral cavity

Warning: do not direct it to your eyes




ECKM1 20 ml oral spray

Included into the **KISS ME KIT**

3E interdenal brush

Interdenal Brush

- Patented, anatomical, scalloped design, for optimal removal of plaque and food debris in interdental areas.
- Nylon bristles with retentive memory (it bends in order to follow the profile and returns to its original shape)
- Ergonomic plastic handle for an easy handling
- Hygienic cap
- Stainless steel Teflon coated wire, anti-scratch, ideal also around titanium implants
- The longer bristles bend over the shorter ones making its insertion easier.

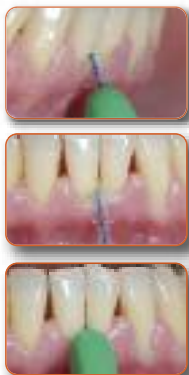

i

Choose the size that best fits the space between your teeth. You can easily insert the brush if you tilt it from the bottom up in the lower jaw, and from the top down in the upper jaw. Several gentle, oblique movements are recommended at least 3 times, in order to remove plaque and food debris.



SMALL
ø from 1,2 to 4 mm
length 10 mm



MEDIUM
ø from 1,6 to 5 mm
length 14 mm



LARGE
ø from 2,2 to 6 mm
length 16 mm



ECE3S 3 small ECE3M 3 medium ECE3L 3 large
ECE6S 6 small ECE6M 6 medium ECE6L 6 large

i small included into the **KISS ME KIT**

IF interdenal floss

Dental Floss

- PTFE, a special ultra-resistant tissue that doesn't fray
- Dental tape, to reach a wider surface



Standard packaging
50 mt

ECIFW 50 mt. white waxed floss



ECIFB 50 mt. grey unwaxed floss



Credit card packaging

with mirror, for a perfect cleaning even outdoors

ECIF1 10 mt. white waxed floss

i
The use of a floss is recommended for any individual with interdental papillas, that guide its insertion. Please remember that the interdental floss is much more effective in the anterior areas.

i ECIF1 included into the **KISS ME KIT**

Enamel Plus

Multipurpose Toothpaste

Low abrasion formula (25 ADA)

- It protects your teeth and restorations
- Delicate taste: no gum irritation
- Strengthens enamel
- Desensitizing properties
- Mild bleaching



ECEP 75 ml.

i
Use with manual or electric toothbrush: make a vertical movement from gum to tooth, on the upper and lower teeth. Repeat the movement also in the inside of the arch.

TS tooth stain remover

Stain remover rubber

- It quickly removes tobacco, coffee and tea stains
- No need to use with local paste



Point shape between teeth



Bullet shape on flat surfaces

i
Run the rubber on the tooth several times until the stain is reduced. Use the point shape between the teeth and the bullet shape on the tooth surface.



ECTS 1 pc.

PW protective wax

Protective wax and silicone

Ideal for orthodontic brackets

- Easy application on orthodontic brackets and wires
- Available in wax or silicone

i
Stick a small piece of orthodontic wax or silicone on the bracket or on the wire. Dry the area before application.



ECPW1 3 silicone bars




ECPW 5 wax bars



OM oral mirror

Oral mirror to check the less visible areas of the oral cavity

- Practical and handy
- To check the hidden areas of the oral cavity and help good oral hygiene

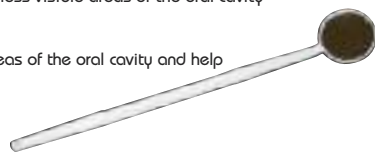

ECOM 1 pc.

