ENA WHITE 2.0

The fastest home professional bleaching

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GB INSTRUCTION REFILL 5 ml

- When the patient comes back for a check after 20 days, check the colour (the patient must bring the empty toothbrush). In case the patient requires further bleaching effect, proceed as follows:
- Take the syringe with 5 ml from the fridge and remove the cap.
- Open the patient's toothbrush after removing the cap and unscrew the head from the dispenser. Rotate the ring towards the direction "DOWN" until the piston reaches the bottom of the dispenser (pic. 1).
- Push the silicon disc of the dispenser using the syringe tip (pic. 2) until it reaches the bottom of the dispenser (pic. 3).
- Inject the gel into the dispenser until the content of the syringe finishes; always leave the syringe tip into the gel in order to avoid any bubbles (pic. 4).
- Screw the same head (which contains the residual gel) on the dispenser and give the toothbrush back to the patient.

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Pic 1. In case further bleaching is needed, the patient must bring the toothbrush to the dentist. Remove the head (see Pic. 2) and rotate the ring towards the direction "DOWN"

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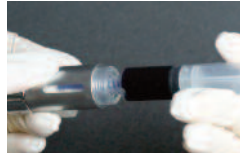
Fig 3. ...verso il fondo del serbatoio

Pic 3. ...towards the bottom of the dispenser

Fig 3. ...hacia el fondo del dispensador

Fig 3. ...vers le fond du dispenser

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Pic 4. Fill it with the gel until the content of the syringe finishes. Screw the head (see Pic. 6) and give the toothbrush back to the patient.

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