TEMP RED

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Indicazioni

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(GB) INSTRUCTIONS FOR USE

Indication

Modelling resin for fabricating secondary constructions with the brushing technique.
Fixation for soldering of e. g. bridges, anchors, attachments, post and core etc.

Contraindication:

If the user is known to be allergic to one of the constituents of this material, it must not be used.

Side-effects:

None known to date.

Properties:

- Burns out completely.
- Minimal shrinkage.

Processing:

- Dispense the required amounts of liquid and powder into separate mixing vessels.
- Dip the tip of the brush into the liquid and then pick up a small amount of powder.
- The resin swells on the brush tip and can be placed and shaped exactly on the coping.
- Repeat this procedure several times until the outer coping has been built up.
- Carefully lift the pattern from the inner coping and check the marginal fit as well as the inside of the pattern.
- Replace the pattern on the inner coping and use a cutter to trim it to uniform thickness.
- After investing, put in oven at 500-700°C (930-1290°F) and proceed to casting.

Please note:

- Powder and liquid can also be mixed by 10:5 ratio.
- If this material is being used on a plaster model, a separating agent should be applied.
- Clean the brush with Temp Red liquid.
- Do not add already used powder and liquid into the bottle.
- Colour: red.

Storage:

Store at room temperature, not more than 30°C (86°F). Do not expose to direct sunlight.

Presentation: 100 gr. powder, 100 ml. liquid, 1 Set: 100gr.powder + 100ml. liquid

Danger:

Do not allow unpolymerized material to contact your skin and do not inhale monomer fumes.

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MICERIUM S.p.A.